## MODE D'EMPLOI

### MANUALE

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### MANUAL DE USO

## DO NOT RE-USE

REF F-06 STERILE R

CE 0123

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## BEDIENINGS- HANDLEIDING

CE 0123

**EN** DO NOT USE THE LENSES IF THE PACKAGING IS DAMAGED. USE THE LENSES ONLY FOR CLINICAL ELECTRORETINOGRAPHY.

Human use of the ERG-Jet requires equipment that complies with all applicable medical device safety standards, to include IED 60601-1. (Please see the connection scheme)\*

### INTENDED USE

The ERG-Jet is used to support diagnostic systems for anomalies of the eye by transmitting the retinal electrical signal.

### CONTRAINDICATIONS

ERG-Jet is not intended for use on children or adults with cases of trauma or disease of the anterior segment of the eye. In the case of ocular and corneal pathologies, its use requires the consent of an ophthalmologist.

### WARNINGS AND PRECAUTIONS

For small children, due to the risk of irritation of the cornea, it should be ensured that they are kept in a stable position for the duration of the test. Eyelid retractor may be used if required. Patients are recommended not to blink with the eye during the test. The manipulation of the ERG-Jet is identical to that of a corneal lens requiring the respect of the rules of the art. In particular the cleanliness in the handling of this sterile object and the unique use.

Mydriatic drops could be used to dilate the pupils for better light response of the eye to the stimuli. This lens should be placed and removed gently. If pain occurs or persists a few hours after the handling, ophthalmological control is considered necessary.

Redness and mild irritation of the eye, without exacerbation of the condition, is possible after performing the procedure. It is suitable to fill the inner concavity of the contact lens with a protective gel prior posing the electrode on the cornea. Blurred vision could be witnessed during the testing when using protective gel. It is suitable to keep the patient in supine position during the test.

In addition of the usual placement process of a conventional contact lens, a topical cornea anaesthesia is recommended prior testing, but general anaesthesia should be avoided, when possible, because it could have an impact on the ERG recordings. The ERG-Jet is not intended for multiple use. If re-used, an infection of the cornea can occur, and the measuring results are no longer guaranteed. The ERG-Jet is not indicated to be used more than one hour. Feeling of discomfort could be witnessed during the test

### SIDE EFFECTS

To date, no undesirable side effects have been observed. However, it may be that a slight unease is experienced during the procedure and the product triggers a slight reddening of the eye. In the case of a persisting impairment, an ophthalmologic control is necessary.

### PROPERTIES

#### 1- OPTICAL PROPERTIES

Transmission of the complete visible light spectrum

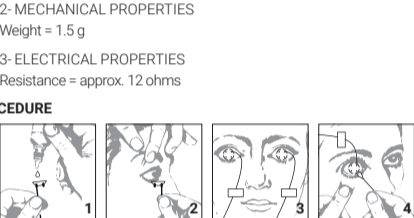
#### 2- MECHANICAL PROPERTIES

Weight = 1.5 g

#### 3- ELECTRICAL PROPERTIES

Resistance = approx. 12 ohms

## PROCEDURE

- 1 Anesthetize the cornea. Hold the concave surface of the ERG-Jet lens upwards and fill it to slightly below the rim with a 2-3% sterile methylcellulose solution.
- 2 Set ERG-Jet lens onto the cornea by the same procedure used for a conventional contact lens. First, lift the patient's upper lid and have him/her look slightly downwards. Gently rest the lens front edge (with the concave surface upwards) on the lower lid, then tilt the lens backwards onto the cornea.
- 3 Attach the connection wire to the patient's face.
- 4 To remove the lens, pull the upper eyelid upwards, then pull the lead wire gently upwards to lift the lower edge of the lens out of the eye.

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