DO NOT RE-USE

MODE D'EMPLOI

REF TC-01 STERILE R

FR

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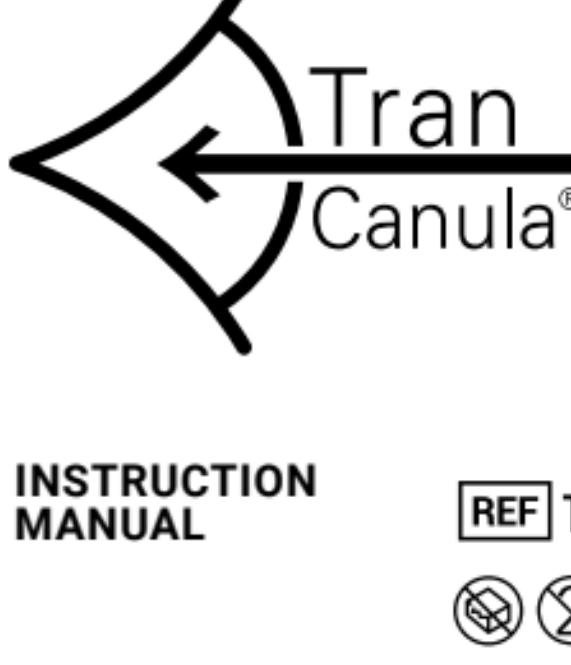
REF TC-01 STERILE R

EN

**DESCRIPTION AND INTENDED USE
OF TRAN CANULA®**

Tran Canula® is a single-use irrigation cannula for washing the anterior chamber of the eye, especially the trabecular meshwork and the iridocorneal angle following cataract surgery.

Tran Canula® is compatible with the Goniowash washing technique.

PRECAUTIONS AND RECOMMENDATIONS

Please read the instructions below carefully and follow the safety precautions.

- In any case, the physician should only operate after having read all the documentation accompanying the product, including instructions for use.
- Only a surgeon specialised in ophthalmology is allowed to use Tran Canula®.
- The use of Tran Canula® and the results obtained are under the sole responsibility of the physician.
- Only the physician is competent to evaluate the relevant indications and contraindications of the technique to be employed.
- Tran Canula® is designed for single use and cannot be reused or resterilised.
- Discard Tran Canula® immediately after use in the medical waste container.
- Do not use Tran Canula® if the unit package is damaged or torn.
- Keep Tran Canula® away from heat and direct light.
- Wearing disposable sterile gloves is strongly recommended to use the device.

PROCEDURE

1. At the end of cataract surgery, connect the Tran Canula® at the irrigation part of the phacoemulsifier. The BSS should be placed at an average height of 80 cm from the eyeball. Introduce the tip of the cannula into the anterior chamber at the corneoscleral junction.
2. Perform the washing of the anterior chamber of the eye being careful not to come in direct contact with the tissues of the inside of the eye.

At the end of the procedure, follow the precautions of use.

CONTRAINDICATIONS

The use of Tran Canula® is contraindicated for patients with the following conditions:

- Pathology of the uvea: inflammation, trauma, atrophy, neo- or hyper-vascularisation.
- Absolute glaucoma or neovascular glaucoma.
- Ocular herpes affecting the cornea, inflammation of the anterior chamber, of the uvea or of the retina.
- Ocular trauma (e.g.: contusion, cut, burn, haemorrhage in the anterior chamber).

As well as patients who have had an iridectomy or laser iridotomies.

**COMPLICATIONS RELATED TO THE USE
OF THE PRODUCT**

In case of contact between the Tran Canula® and tissues of the interior of the eye the following complications may occur:

- Temporary deformation of the pupil.
- Thinning, atrophy, inflammation or perforation of the iris.
- Iridodialysis.
- Minimal and brief haemorrhage in the anterior chamber.
- Inflammation of the cornea.
- Tearing and detachment of the corneal endothelium.

Moreover, there are also the complications related to cataract surgery:

- Infections (1 to 3 cases out of 1000).
- Eye trauma caused by the patient or their relations.
- Scar insufficiently waterproof.
- Partial ptosis of the upper eyelid.
- Subconjunctival or eyelid haematoma.
- Perceiving flying spots.
- Ocular inflammation.
- Increase in intraocular pressure.
- Astigmatism.

Additional information: fabrinal.com

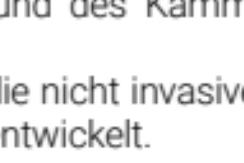
FABRINAL SA, RUE DE LA TUILERIE 42
CH-2300 LACHAUX-DE-FONDS, SWITZERLAND
T +41 32 968 09 08, FAX +41 32 968 42 67
fabrinal.com, info@fabrinal.ch

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