MADE IN CH DO NOT RE-USE

MD

(infection risk)

REF

TC-01

STERILE R

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MADE IN SWITZERLAND

#### INTENDED USE

TranCanula® is a single-use irrigation cannula for washing the anterior chamber of the eye, especially the trabecular meshwork and the irido-corneal angle following cataract surgery. TranCanula® is compatible with the Goniotomy washing technique.

#### INDICATION FOR USE

TranCanula® is indicated for use following cataract surgery combined with Goniotomy in the eyes of patients with observed pseudoexfoliation syndrome.

#### PATIENT POPULATION, USER(S) AND USE ENVIRONMENT

Patients requiring cataract surgery who suffer from pseudoexfoliation syndrome.

TranCanula® is to be used in a surgical setting by professional ophthalmologists trained to conduct cataract surgery.

TranCanula® is designed with a female bore fitting male connectors of BSS dispensers irrigation lines.

#### CONTRAINDICATIONS

The use of Tran Canula is contraindicated for patients with the following conditions:

- Pathology of the uvea: Inflammation, trauma, atrophy, neo- or hyper-vascularisation.
- Absolute glaucoma or neovascular glaucoma.
- Ocular herpes affecting the cornea, inflammation of the anterior chamber, of the uvea or of the retina.
- Ocular trauma (e.g.: Contusion, cut, burn, haemorrhage in the anterior chamber).
- Patients who have had an iridectomy or laser iridotomy.

#### PRECAUTIONS

DO NOT USE THE TRAN CANULA® IF THE PACKAGING IS DAMAGED.

Please read the instructions below carefully and follow the safety precautions.

- In any case, the physician should only operate after having read all the documentation accompanying the product, including instructions for use.
- Only a surgeon specialised in ophthalmology is allowed to use TranCanula®.
- The use of TranCanula® and the results obtained are under the sole responsibility of the physician.
- Only the physician is competent to evaluate the relevant indications and contraindications of the technique to be employed.
- TranCanula® is designed for single use and cannot be reused or resterilised.
- Discard TranCanula® immediately after use in the medical waste container.
- Wearing disposable sterile gloves is strongly recommended to use the device.
- Use the TranCanula® only for washing the anterior chamber of the eye, especially the trabecular meshwork and the irido-corneal angle following cataract surgery (transient use).

Human use of the TranCanula® requires a phacoemulsification device that complies with all applicable medical device safety standards, to include IEC 60601-1.

#### PROCEDURE

- At the end of cataract surgery, connect the TranCanula® at the irrigation part of the phacoemulsifier. The BSS should be placed at an average height of 80 cm from the eyeball. Introduce the tip of the cannula into the anterior chamber at the corneoscleral junction.

- Perform the washing of the anterior chamber of the eye being careful not to come in direct contact with the tissues of the inside of the eye.

- As TranCanula is a single-use device, it must be segregated from reusable devices. Once used, it should be disposed and not be returned to a decontamination facility for reprocessing.

#### SIDE EFFECTS

In case of contact between the TranCanula® and tissues of the interior of the eye the following complications may occur:

- Temporary deformation of the pupil.
- Thinning, atrophy, inflammation or perforation of the iris.
- Iridodialysis.
- Minimal and brief haemorrhage in the anterior chamber (blood reflux and blood in the periphery of the iris).
- Inflammation of the cornea.
- Depigmentation, pigment dispersion.
- Tearing and detachment of the corneal endothelium.

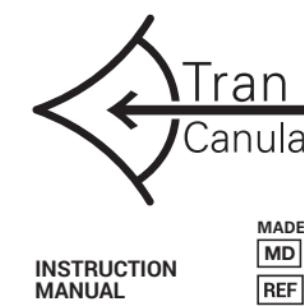
Moreover, there are also the major complications following cataract surgery:

- Retinal detachment.
- IOL dislocation.
- Corneal decompensation.
- Diplopia.
- Blindness.
- Infections (1 to 3 cases out of 1000).
- Eye trauma caused by the patient or their relations.
- Scar insufficiently waterproof.
- Partial ptosis of the upper eyelid.
- Subconjunctival or eyelid haematoma.
- Perceiving flying spots.
- Ocular inflammation.
- Increase in intraocular pressure.
- Astigmatism.

Other complications following cataract surgery:

- Iris trauma.
- Descemet membrane tears or detachment.
- Posterior capsular tear or zonular rupture.
- Retained lens fragments.

For any questions, suggestions, or to report any incident concerning TranCanula®, please get in touch with (and report any incident to the authority having jurisdiction): fabrinol.com

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**MADE IN CH** **DO NOT RE-USE**  
(infection risk)  
**MD** **REF TC-01** **STERILE R**  
 **CE 0123**

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